

Instructions on how to install IntelViewer on a Mac

1. Log In

Visit <https://images.riverradiology.co.nz> and log in to IntelBrowser.

2. Installers

In the IntelBrowser main menu, click Installers.
The Intelrad User Resources page appears

3. IntelViewer Tracks

Click IntelViewer Tracks.

4. New Page

The IntelViewer Tracks page appears.

5. Download

From Stable Track, click Download IntelViewer for the Mac Os X Installer.

6. Double Click

Double-click IntelViewer.dmg
This disk image contains the IntelViewer application and license agreement.

7. Applications

Drag the IntelViewer application to the Applications folder at the top level of the hard disk on your Mac.

8. Double Click

Double-click IntelViewer.

9. Access

To provide quick access to IntelViewer, add it to the Dock.