

Instructions on how to install IntelViewer on a PC

1. Log In

Visit <https://images.riverradiology.co.nz> and log in to IntelBrowser

2. Installers

In the IntelBrowser main menu, click Installers

3. IntelViewer Tracks

Click IntelViewer Tracks

Please refer to the image fig2.

4. Download

From Stable Track, click Download IntelViewer for either the 32-bit Windows Installer or 65-bit Windows Installer depending on what system type your computer is*.

Please refer to the image fig2.

5. Save

When prompted select the option Save to begin the download and save it for installation.

6. Install

Once the download process has completed double click on the installer named IntelViewer-install and follow instructions selecting the default options.

7. Complete

Once the install is complete you will notice an IntelViewer icon on your desktop.

*In order to determine what system type your PC is open the Windows Start menu, right click on Computer and select Properties. This will open a window to view basic information about your computer, it is here that you will be able to find out whether you are running a 32-bit or 64-bit Operating System.